



Purleve Face Mask Instructions

How to apply and wear your mask

Face masks provide protection against large droplets, splashes or sprays of bodily or hazardous fluids. The masks do not provide the wearer with reliable protection from inhaling smaller airborne particles and are not considered respiratory protection.



Center seam is at the bottom of the mask.

Slide the mask over the top of your head. Note: The center seam is at the bottom and fits under your chin.

Tie the bottom strap into a knot to tighten if necessary.



If the bottom strap is a little loose, pull it tight and tie into a knot. NOTE: Toggles can also be used for easy tightening / loosening.



Hook the bottom of the mask under your chin and pull the top strap up to the center of the back of your head.



The mask should fit securely over your nose, the sides of your face and be tight under your chin.

PLEASE CONTACT THE **Pūrieve** TEAM FOR ALL INQUIRIES:

Pūrleve[®] Headquarters 111 W. Olive St. Glendale, WI 53212 PHONE: (407) 935 - 1180 TOLL FREE: (877) 787 - 5383 FAX: (407) 935 - 0001

Pūrleve [™] www.purleve.com