



Made in USA

Purleve™ Face Mask Distribution & Washing Instructions

SAFE DISTRIBUTION GUIDELINES

- Prior to distributing, a designated person should wash their hands with soap and water for 20-30 seconds and put on a new pair of gloves, if available.
- Open package of masks. Remove one of the masks and put it on.
- Avoid touching the front of the mask.
- While maintaining appropriate distancing, distribute masks by holding it by the strap and placing it on the workstation, desk or counter nearby.
- After distributing masks the remaining face masks should be kept in the plastic packing bag.

WASHING INSTRUCTIONS

- Machine or hand wash with warm water and mild detergent. Make sure it does not contain bleach, or has a color safe bleach (Tide Cold Water).
- Tumble dry low/medium heat or air dry.

PLEASE CONTACT THE **Purleve** TEAM FOR ALL INQUIRIES:

Purleve™ Headquarters
111 W. Olive St.
Glendale, WI 53212

PHONE: (407) 935 - 1180
TOLL FREE: (877) 787 - 5383
FAX: (407) 935 - 0001

Purleve™
www.purleve.com