



Pūrleve[™] Form Fitted COMBO Mask

(Neck & Ear Elastic Design)



STEP ONE:

Put the elastic band over your head so that the elastic wraps around your neck and the mask falls at your chest.



STEP TWO:

Put your fingers into the ear loops and adjust the loops so there is ample space to wrap around your ears.



STEP THREE:

Pull the mask up to your face and attach it around your ears.



STEP FOUR:

Using the Toggle on the elastic around the neck, adjust the tension so that the mask fits securely and comfortably around your face.

PLEASE CONTACT THE **Pūrleve** TEAM FOR ALL INQUIRIES: